

# Activ Endermology Solo

Manual guide



## Metrics

---

### Producer

#### **VACU ACTIV**

##### **World Leading Manufacturer**

Łosino, ul. Modrzewiowa 1  
76-251 Kobylnica  
POLAND

##### **Poland**

+48 059 843 43 43

##### **World Wide**

+48 660 216 370

##### **E-mail**

[contact@vacuactiv.com](mailto:contact@vacuactiv.com)

### **Customer service / Technical service (ordering spare parts for machine components):**

[www.vacuactiv.com/technical-support/](http://www.vacuactiv.com/technical-support/)

##### **E-mail**

[support@vacuactiv.com](mailto:support@vacuactiv.com)

##### **Phone**

+48 501 186 938

**[www.vacuactiv.com](http://www.vacuactiv.com)**

# Contents

---

<b>1. Accessories connection</b>	<b>4</b>
<b>2. Operation instruction</b>	<b>7</b>
<b>3. Magic Curve operation instruction</b>	<b>33</b>
<b>4. Anti-Cellulite operation instruction</b>	<b>35</b>
<b>5. Solution with simple Malfunction</b>	<b>37</b>
<b>6. Specification</b>	<b>38</b>
<b>7. Pcking list</b>	<b>39</b>
<b>8. Warm tips</b>	<b>40</b>
<b>9. Operation cautions</b>	<b>41</b>
<b>10. Contact</b>	<b>43</b>

# 1. Accessories Connection

---

## 1. Magic Curve Handle connection

1. Magic curve handle and Magic curve heads



2. Align the handle to the Magic curve head




3. Fix the side face plastic clips



4. Open the two sides bottom of the plastic clips if need to change the Magic curve head.




## 2. Anti-cellulite handle connection

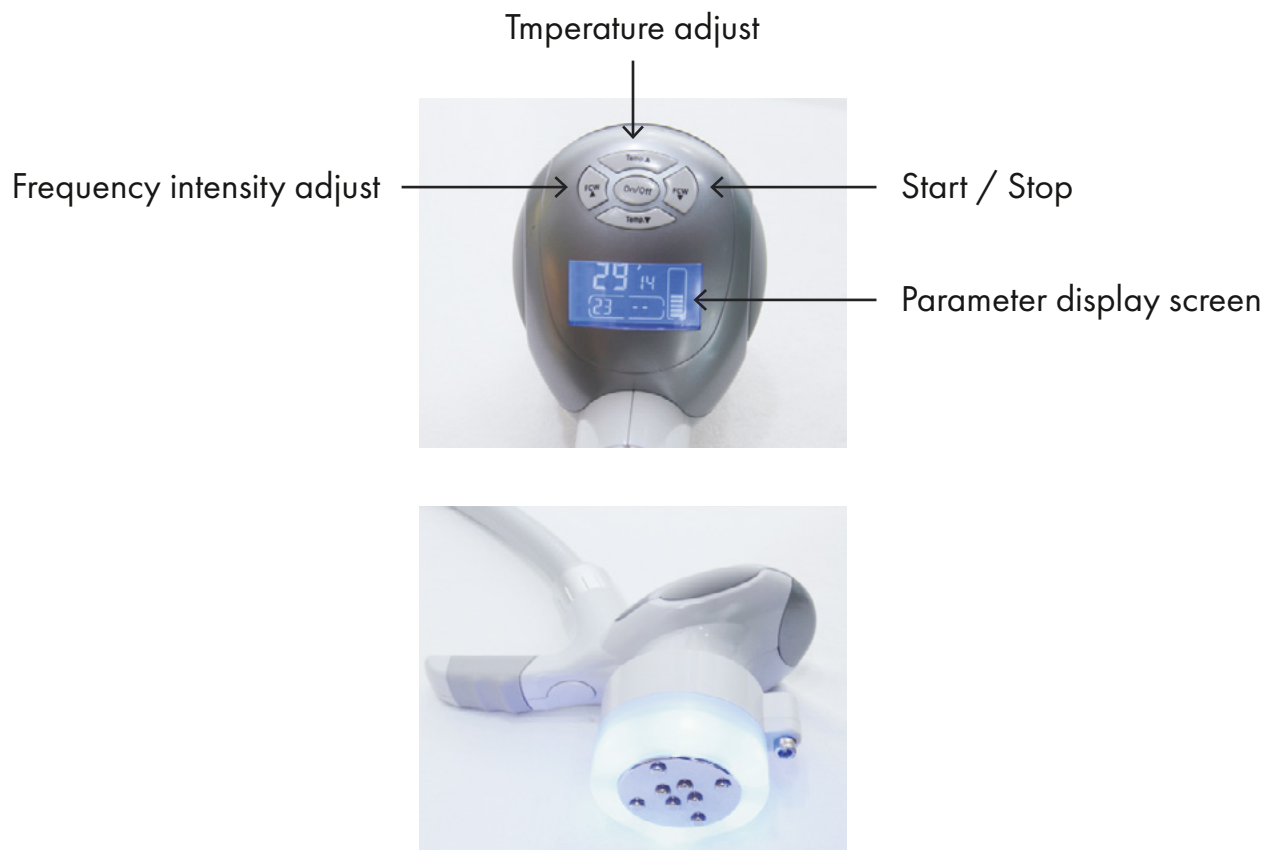

The Anti-Cullulite Handle no need teardown, After adjust the parameter, Use the Jel while using the handle

### **3. Massage Cloth(Use it while operating the Magic Curve heads)**

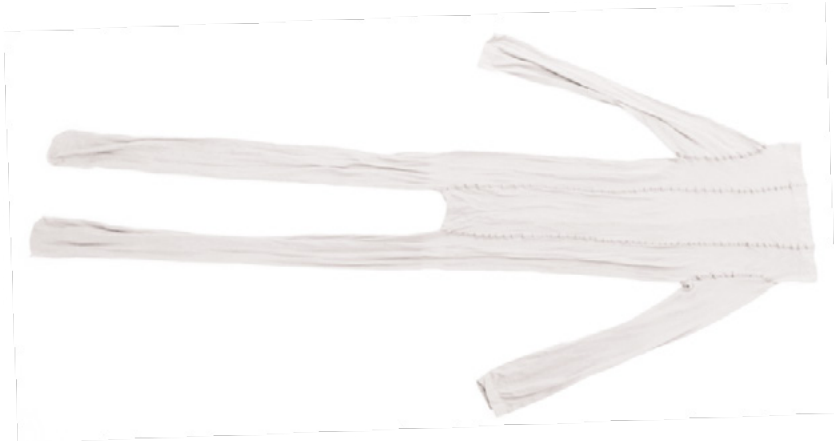

## 2. Operation instruction

### 1. Homepage operation instruction

Drain water  
key



Sounds  
adjustment

TREATMENT  
SELECTION

Anti-  
Cellulite  
operation  
instruction




TREATMENT  
SELECTION


Magic  
Curve



## 2. Anti-Cellulite Panel instruction

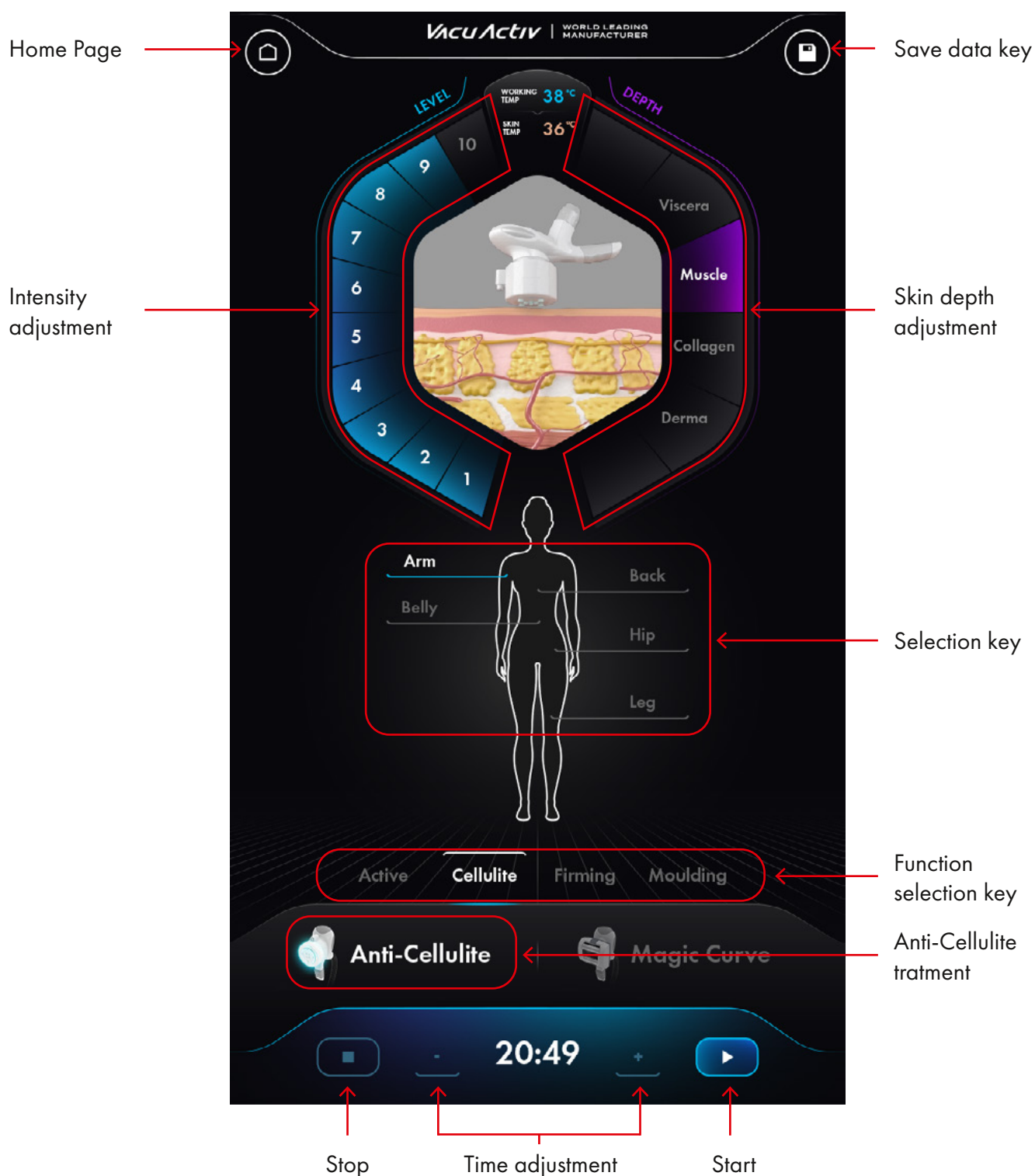

### 3. Magic Curve operation instruction

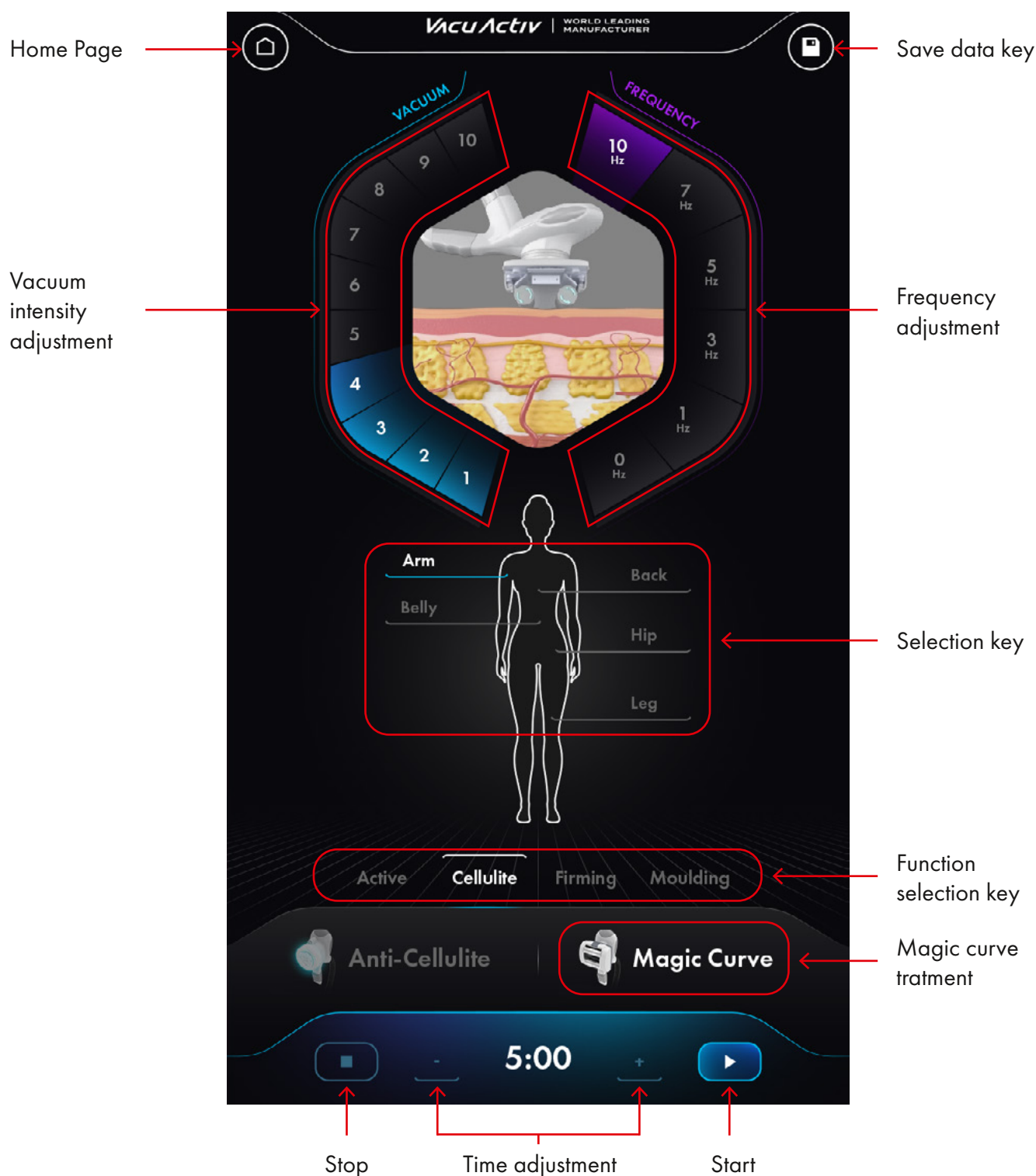

## Homepage operation instruction

Connect the power wire , start the Power, The screen display the Homepage

Drain the useless water, After connect the drain pipe, press the homepage ,Pop-up window, select "Yes"




## Operation steps for Magic Curve



### Belly

Step 1 : Relaxing

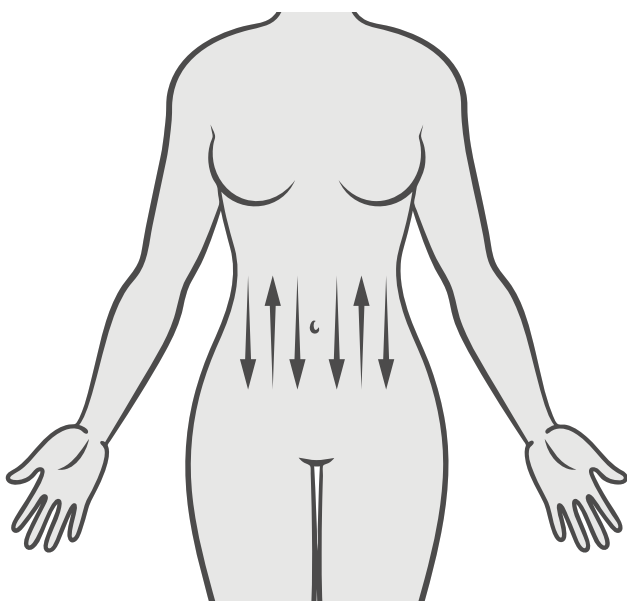

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment  
**Magic Curve**

Part of the body  
**Belly**

Time  
**5:00**

Vacuum  
**10 level**

Mode  
**Active**

Frequency  
**10Hz**


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Belly

#### Step 2: Cellulating

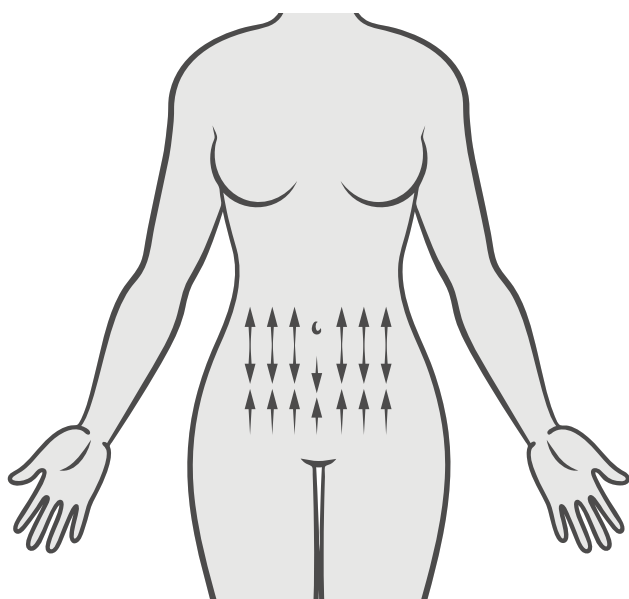

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Belly</b>
Time	Vacuum
<b>20:00</b>	<b>5 level</b>
Mode	Frequency
<b>Cellulite</b>	<b>1Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Belly

#### Step 3: Wellness

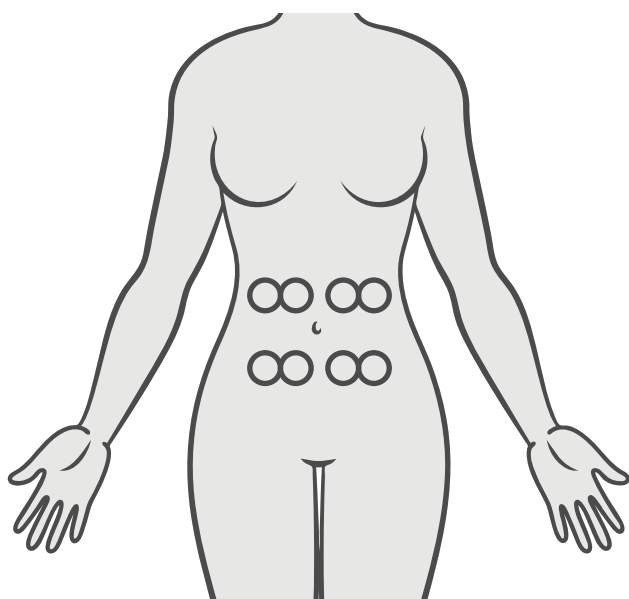

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Belly</b>
Time	Vacuum
<b>5:00</b>	<b>3 level</b>
Mode	Frequency
<b>Moulding</b>	<b>0Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Belly

#### Step 4: Firming

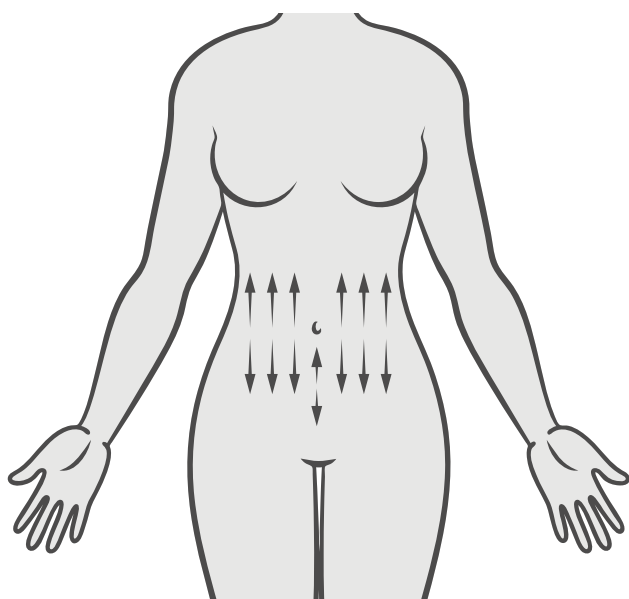

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Belly</b>
Time	Vacuum
<b>5:00</b>	<b>5 level</b>
Mode	Frequency
<b>Firming</b>	<b>1Hz</b>


#### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Back

#### Step 1: Relaxing

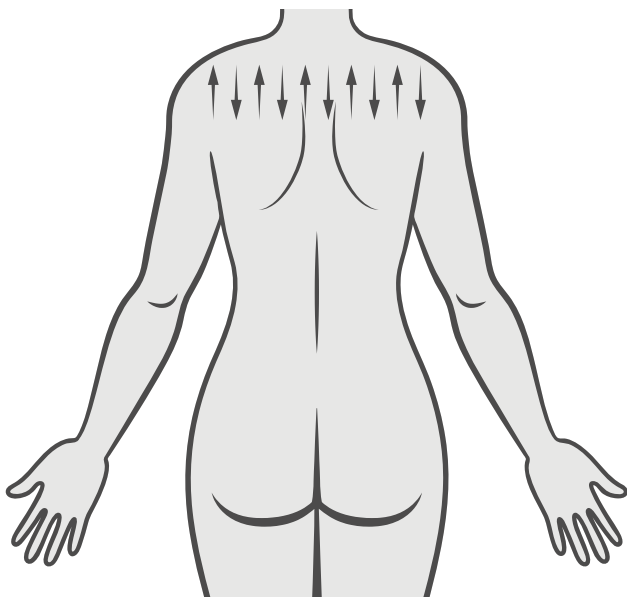

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Back</b>
Time	Vacuum
<b>5:00</b>	<b>10 level</b>
Mode	Frequency
<b>Active</b>	<b>10Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Back

#### Step 2: Cellulating

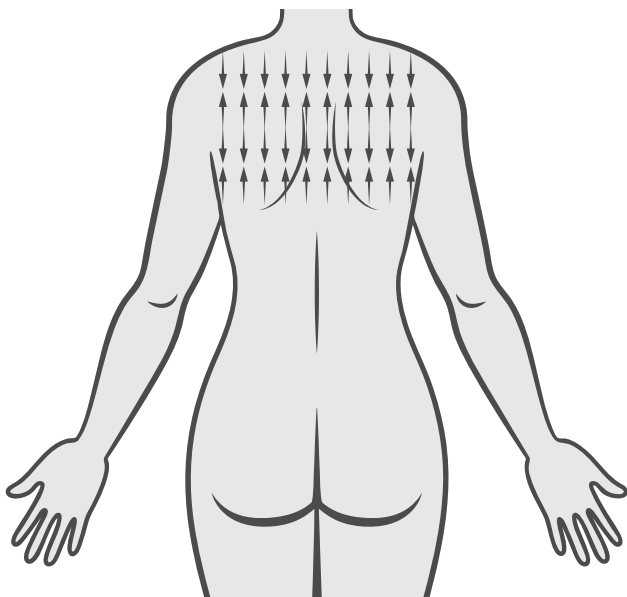

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Back</b>
Time	Vacuum
<b>20:00</b>	<b>5 level</b>
Mode	Frequency
<b>Cellulite</b>	<b>1Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Back

#### Step 3: Wellness

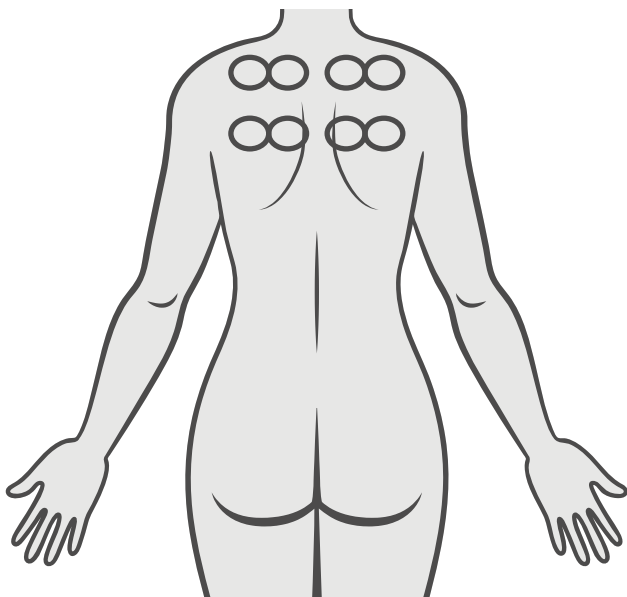

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Back</b>
Time	Vacuum
<b>5:00</b>	<b>3 level</b>
Mode	Frequency
<b>Moulding</b>	<b>0Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Back

#### Step 4: Firming

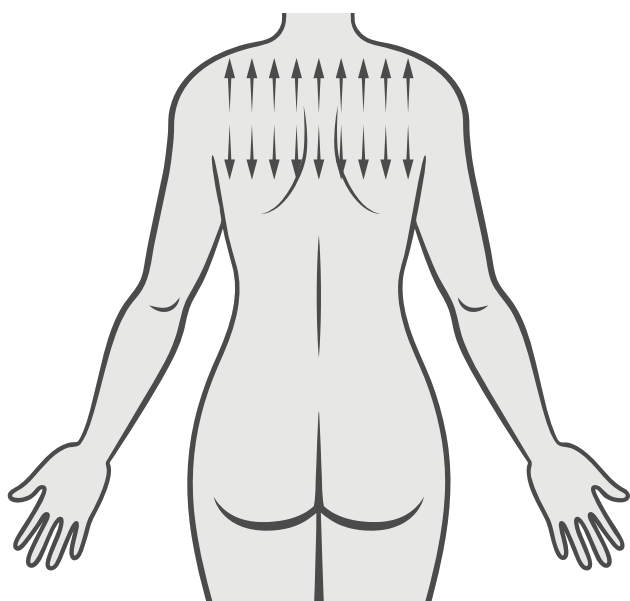

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Back</b>
Time	Vacuum
<b>5:00</b>	<b>5 level</b>
Mode	Frequency
<b>Firming</b>	<b>1Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Hip

#### Step 1: Relaxing

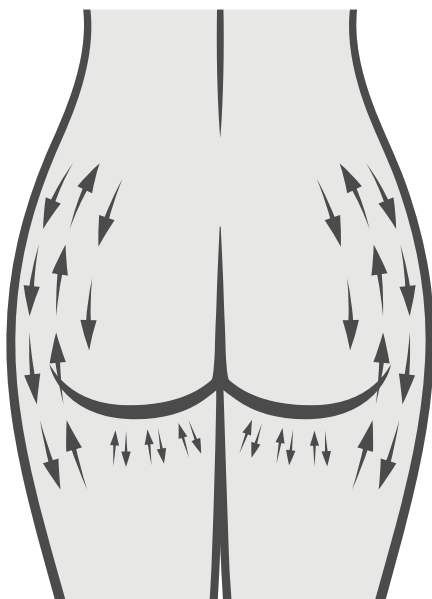

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Hip</b>
Time	Vacuum
<b>5:00</b>	<b>10 level</b>
Mode	Frequency
<b>Active</b>	<b>10Hz</b>

### Note:



The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Hip

Step 2: Cellulating

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

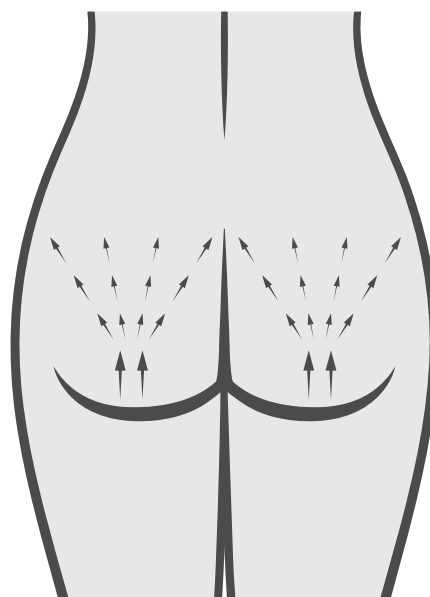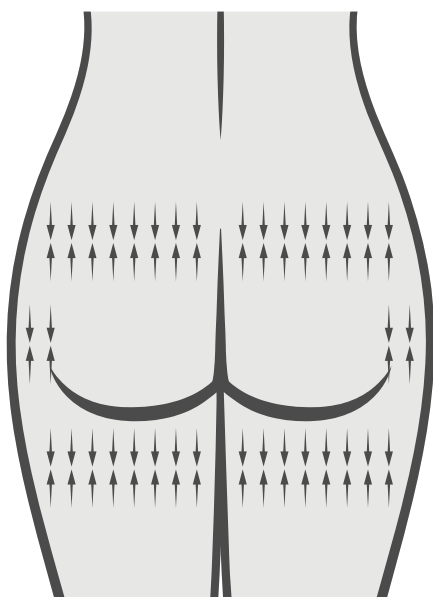

Treatment settings:

Treatment  
**Magic Curve**

Part of the body  
**Hip**

Time  
**5:00**

Vacuum  
**10 level**

Mode  
**Active**

Frequency  
**10Hz**


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Hip

#### Step 3: Wellness

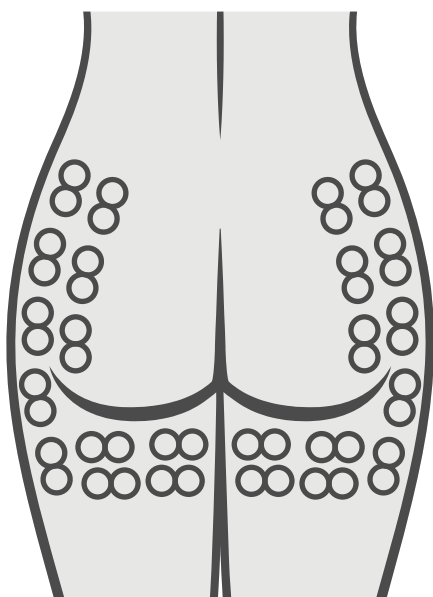

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Hip</b>
Time	Vacuum
<b>5:00</b>	<b>3 level</b>
Mode	Frequency
<b>Moulding</b>	<b>0Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Hip

#### Step 4: Firming

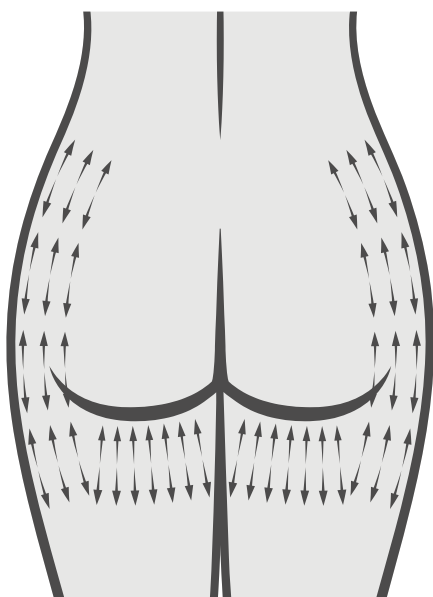

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Hip</b>
Time	Vacuum
<b>5:00</b>	<b>5 level</b>
Mode	Frequency
<b>Firming</b>	<b>1Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Legs

#### Step 1: Relaxing

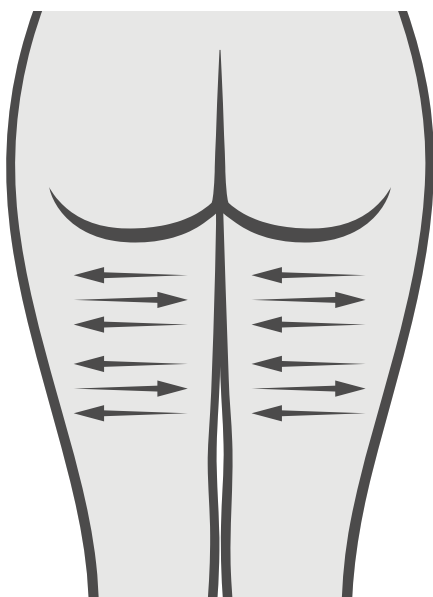

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Legs</b>
Time	Vacuum
<b>5:00</b>	<b>10 level</b>
Mode	Frequency
<b>Active</b>	<b>10Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Legs

#### Step 2: Cellulating

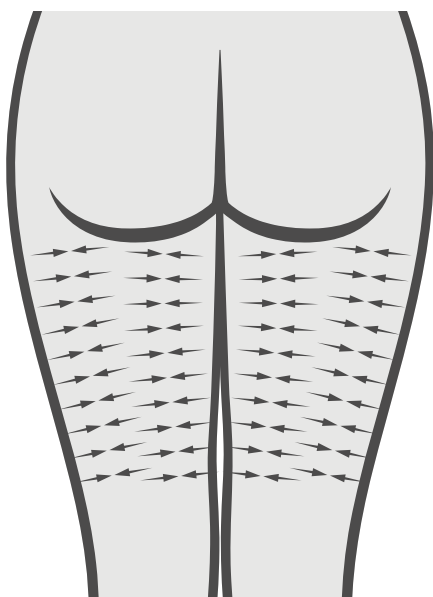

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Legs</b>
Time	Vacuum
<b>10:00</b>	<b>5 level</b>
Mode	Frequency
<b>Cellulite</b>	<b>1Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Legs

#### Step 3: Wellness

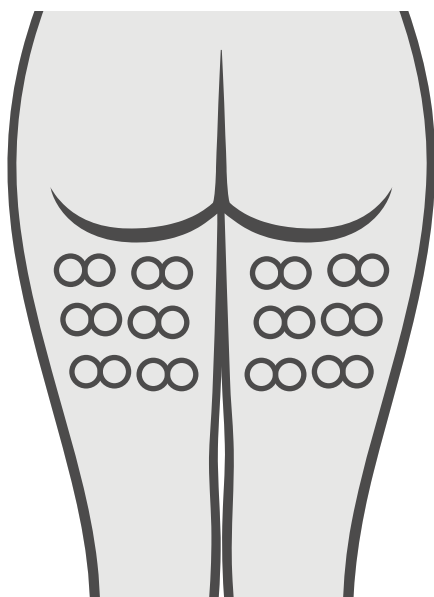

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Legs</b>
Time	Vacuum
<b>5:00</b>	<b>3 level</b>
Mode	Frequency
<b>Moulding</b>	<b>0Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Legs

#### Step 3: Firming

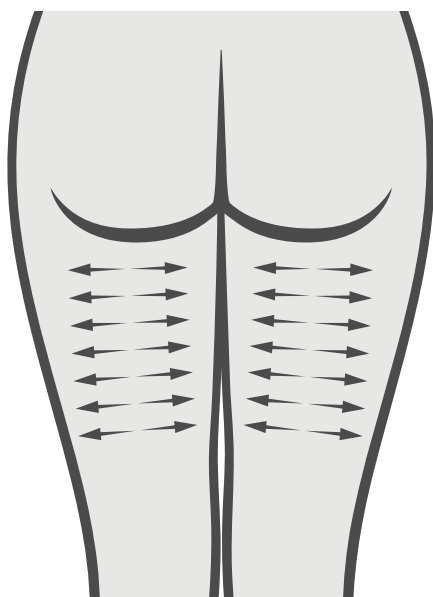

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Legs</b>
Time	Vacuum
<b>5:00</b>	<b>5 level</b>
Mode	Frequency
<b>Firming</b>	<b>1Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Arms

#### Step 1: Relaxing

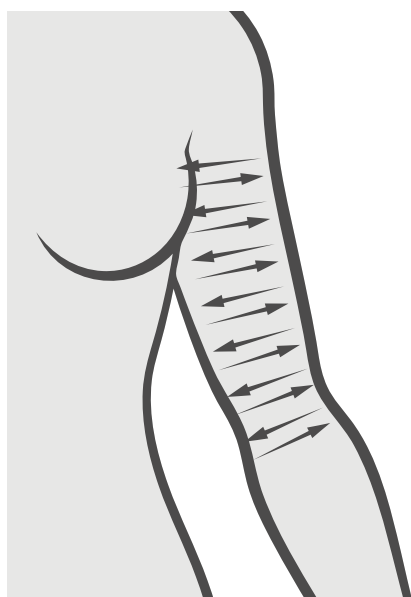

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Arm</b>
Time	Vacuum
<b>5:00</b>	<b>10 level</b>
Mode	Frequency
<b>Active</b>	<b>10Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Arms

#### Step 2: Cellulating

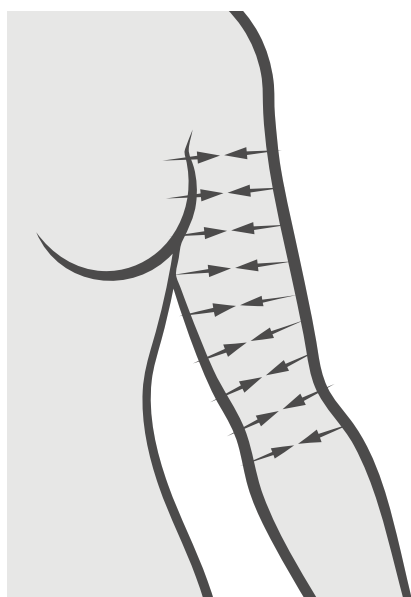

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Arm</b>
Time	Vacuum
<b>10:00</b>	<b>5 level</b>
Mode	Frequency
<b>Cellulite</b>	<b>1Hz</b>

### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Arms

#### Step 3: Wellness



Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Arm</b>
Time	Vacuum
<b>3:00</b>	<b>3 level</b>
Mode	Frequency
<b>Moulding</b>	<b>0Hz</b>


### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

## Operation steps for Magic Curve

### Arms

#### Step 4: Firming

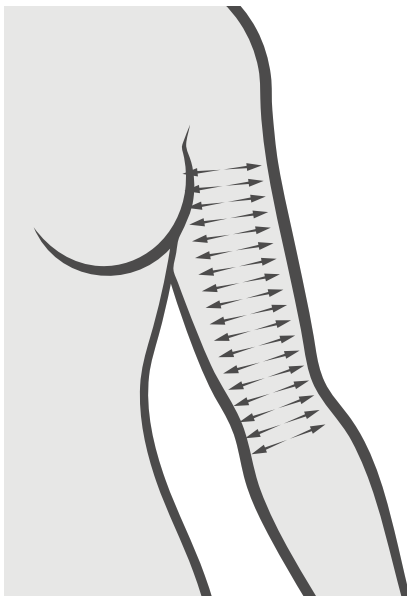

Avoid the Magic Curve roll through the navel.  
If the belly feel hard and fever, please stop the treatment and consult your doctor.

Treatment settings:

Treatment	Part of the body
<b>Magic Curve</b>	<b>Arm</b>
Time	Vacuum
<b>5:00</b>	<b>5 level</b>
Mode	Frequency
<b>Firming</b>	<b>1Hz</b>

### Note:

The above data is for reference only , please communicate with client during treatment and adjust the data according to client's actual feeling .

# Operation steps for Anti-Cellulite



Operation Steps for Anti-cellulite

## Belly

Step 1 : Adjust the parameter as following (only have "All-round" mode):

Treatment settings:


Treatment	Part of the body
<b>Anti-Cellulite</b>	<b>Belly</b>
Time	Level
<b>15:00</b>	<b>5</b>
Mode	Depth
<b>All-round</b>	<b>Collagen</b>

Step 2: Smearing some cream or aesthetic medium on the body part where need to treatment.

## Operation steps for Anti-Cellulite

---

Step 3: Operating practices as following ---- probe do spiral movement along the belly, during working, have to keep move, forbid to stay in ame area, prevent scald client.

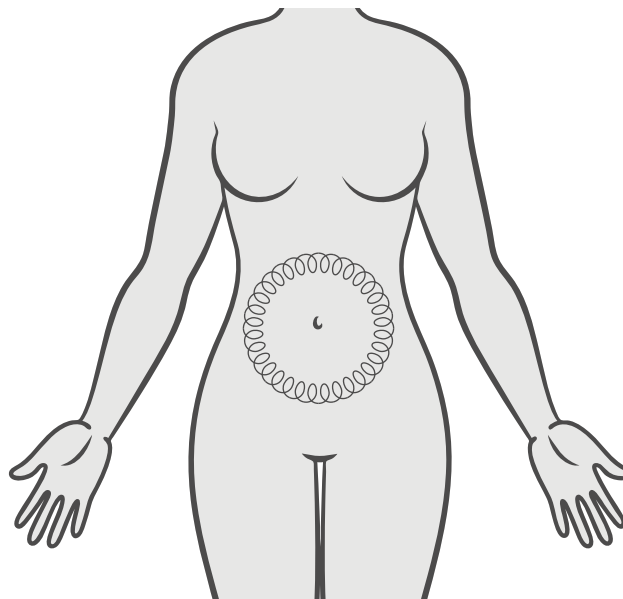**Note:**

Before use the probe , please use the funnel put some water into the machine ; if it is filled , the water will backflow from the machine . And than open the machine , have to let the probe's temperature drop to about 5°-10°,after that start to work on client's body .





### 3. Magic Curve Operation instruction

- 1 Press the Homepage, select the Magic Curve, Enter the operation interface, adjust the parameter  
 Step 1: Select the Part (Arm/Back/Abdomen/Buttock/Leg)  
 Step 2: Select the Function(Active/Cellulite/Firming/Moulding)  
 Step 3: Adjust the Vacuum Intensity(According to different customer)  
 Step 4: Adjust the Frequency intensity  
 Step 5: Setting the time

- 2 Remark: The Magic Curve have 4 Modes: Active, Cellulite, Firming, Moulding





- 3 Setting the mode:

Four modes used for different Parts			
Active	Cellulite	Firming	Moulding
 <p>2 kinds of modes: The roller turn to left or turn to right</p>	 <p>1 kind of Mode: The Roller Turn inward</p>	 <p>1 kind of Mode: The Roller Turn outward</p>	 <p>2 kinds of modes: The roller turn to left or turn to right</p>
Function: Relaxing, Dredge lymph	Function: Celluliting, Lose weight	Function:Firming the Muscle	Function: Stimulate collagen synthesis. Sculptural body curve

- 4 After setting the parameter , if you want to save the data , Press the top right icon, pop-up window,select "Yes",Then save the data

Icon Save the Data

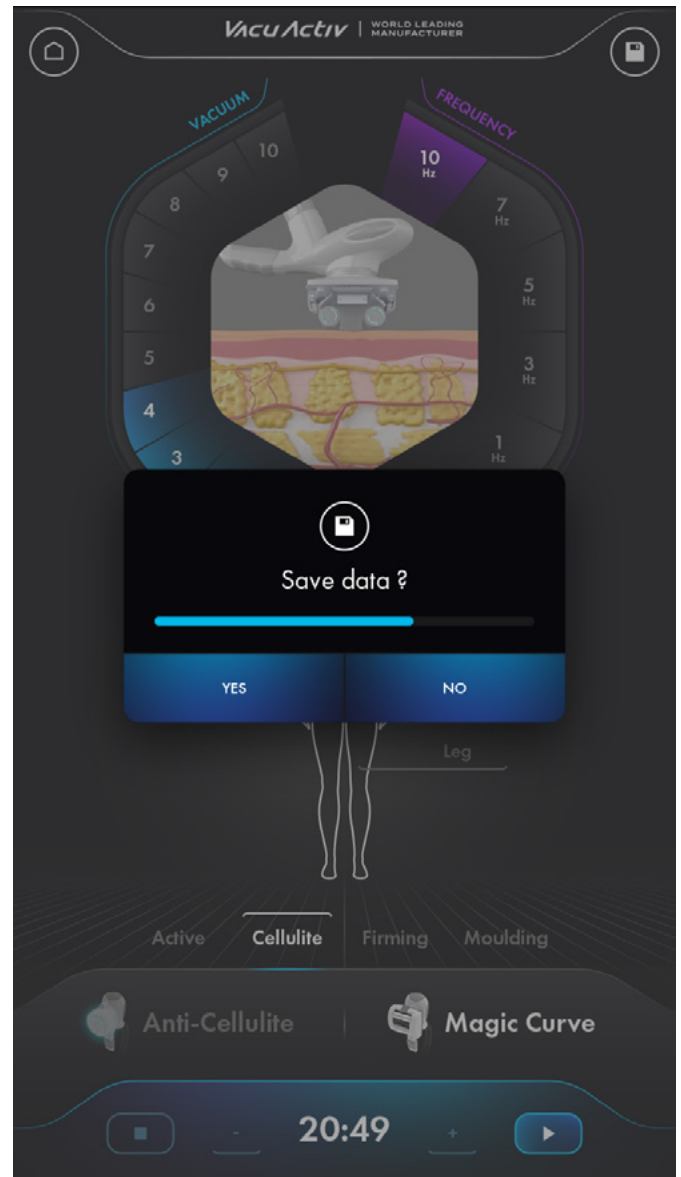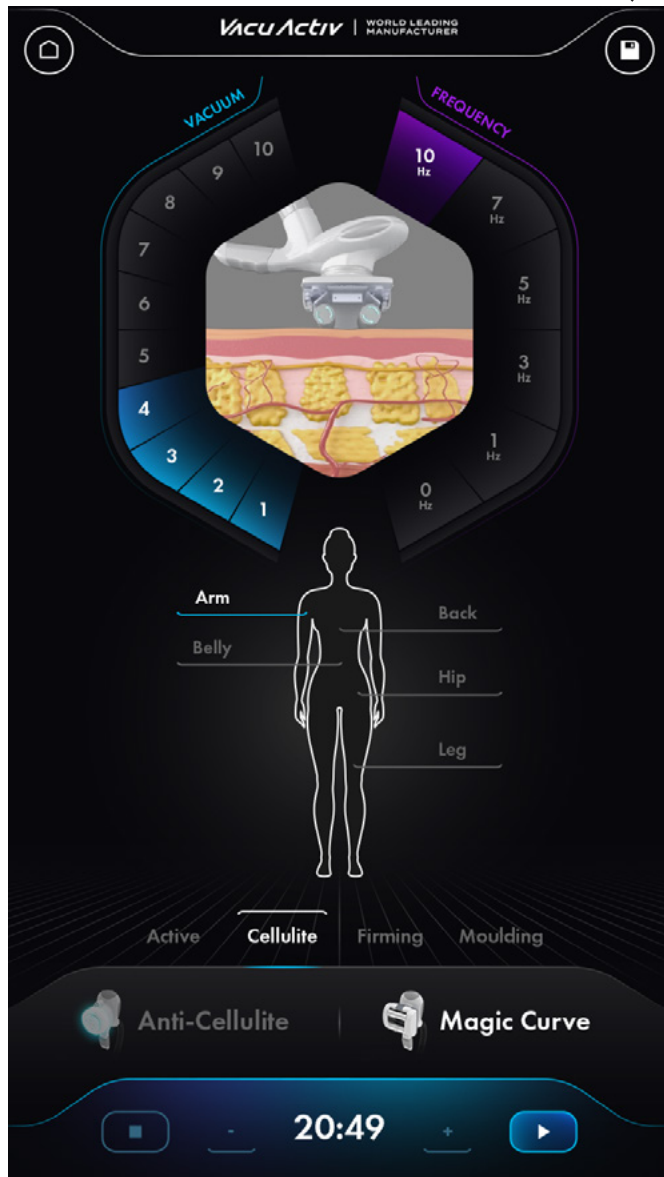

- 5 After setting , Press the "Start",begin to work , If need to revise the parameter, Press the "Stop",Then revise it

## 4. Anti-Cellulite Operation instruction

---

- 1 Press the Homepage, select the Anti-cellulite, enter the operation interface, setting the parameter  
Step 1: Select the Part (Arm/Back/Abdomen/Buttock/Leg)  
Step 2: Select the Function (Slimming/Shape/Firming/All-Round)  
Step 3: Adjust the Level (According to different customer)  
Step 4: Adjust the Depth
  
- 2 Adjust the Temperature on The Anti-Cellulite Handle



Increase Temp.



Decrease Temp.

### **Remark:**

When the temp display "--", It means Stop cooling, Normally No need cooling.

- 3 After setting the parameter , if you want to save the data, Press the top right icon, pop-up window, select "Yes", Then save the data

Icon Save the Data

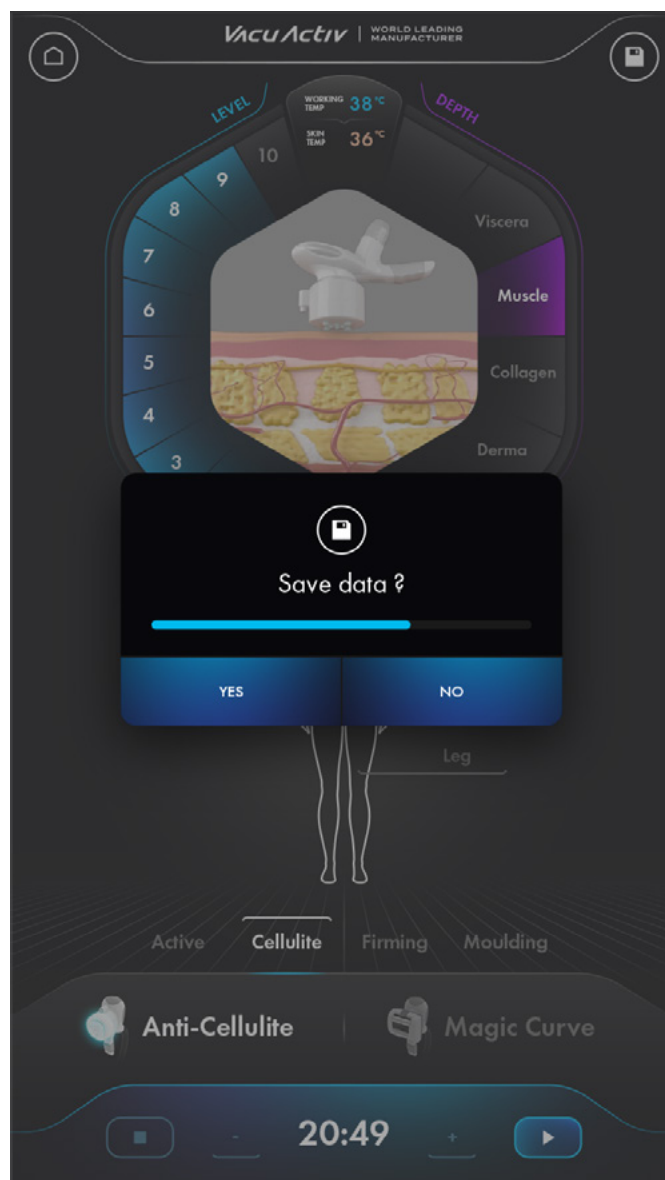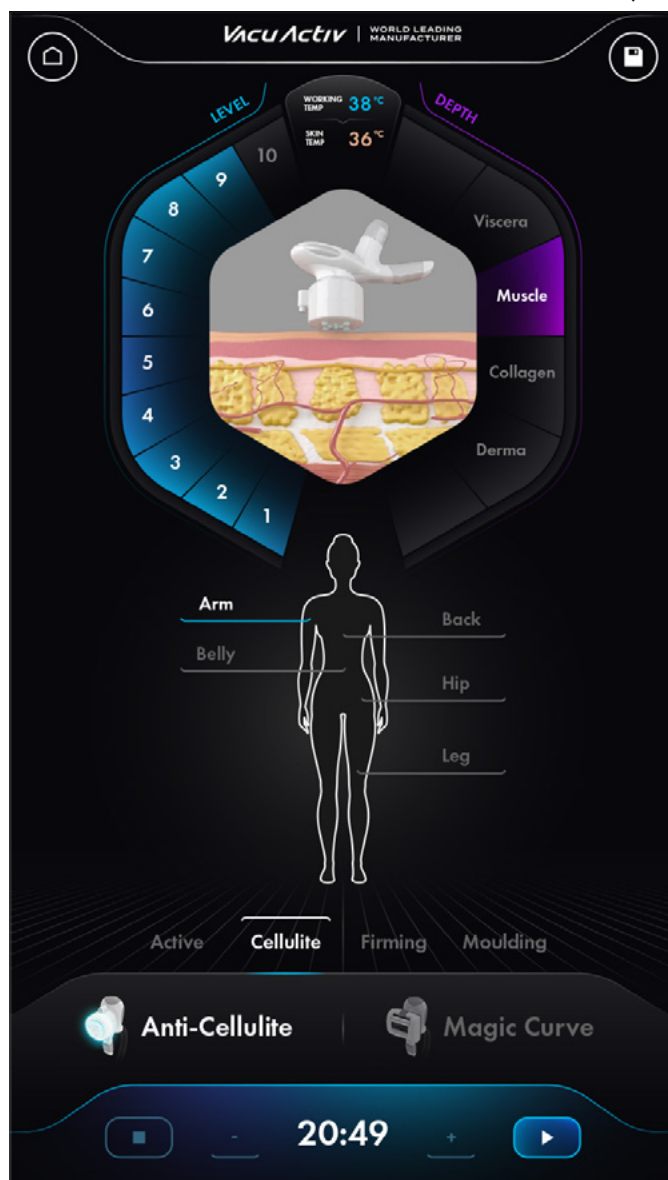

- 4 After setting , Press the "Start",begin to work , If need to revise the parameter, Press the "Stop",Then revise it

## 5. Solution with simple Malfunction

Failure	Solution
Power on, the machine no reponse	Check whether the power wire is in good connection with mainframe
	Check whether the power switch is on
Small suction no suction	Inspect whether the air line connected well with the main frame whether the main frame displays good
	Inspect whether the filter has not been cleaned for a long time or the filter screen is blocked Whether the inflator is broken or air leakage
During the operation ,the machine is over-heat	Using for too long time, turn it off and let it naturally cool down
	Check whether the power is OK (inspected by technician)
No power	Check the fuse, if it is broken, replace it

## 6. Specification

<b>MEAS</b>	<b>Height</b> 168,1 cm (66,1 inches) <b>Height 2</b> 83,9 cm (33,03 inches) <b>Width</b> 56,8 cm (22,3 inches) <b>Depth</b> 96 cm (37,8 inches)
<b>Power</b>	750W
<b>Voltage</b>	220-230V
<b>Electric current</b>	10A
<b>Technology</b>	RF, negative pressure, massage with self-propelled rollers
<b>Underpressure</b>	690mbar
<b>Radio frequency</b>	500 Hz
<b>Working frequency</b>	0-20Hz

## 7. Packing list

ITEM Activ Endermology Solo				
Name	Unit	QTY	MEAS	Remark
Main	set	1		
Magic curve head	pc	2	One smooth head, One Treaded heand	
Magic curve handle	pc	1		
Anti-cellulite Handle	pc	1		
Massage cloth	Set	10		
Power wire	strip	1		
Seal	pc	2		
Spanner	pc	1		
Add water funnel	pc	1		
Accessory Holder	pc	1		
Filter	pc	2		
Fuse	pc	4		
Instruction	pc	1		
Accessotory box	pc	1		
Silicon Pipe	strip	1		

## 8. Warm tips

---

### 1. Some matters need attention when Use Magic Curve handle

Before use the machine, please read carefully the following information:

- If skin have rebefaction or have scar while using the machine , please stop operation immediately! If feel pain while using , please tell the operator reduce the suction in time .
- When operation the machine , Do not keep the handle on the same part for a long time , Remove the handle on body or on face slowly .
- The children which under 7 years old are forbidden .If the Children over 7 years old need use the machine , Need doctor's advice
- People who Have a fever,Infection ,Inflammation,skin disease (sAcne,Eczema,Herpes,HPV),Fresh Scab,Serious or chronic disease (Cancer,AIDS,Blood disease )are Forbidden.
- If you have a risk of Phlebitis, and had vein surgery within 2 months , Probibition of use the machine .If have any questions,please make advice from your doctor
- Pregnant women are forbidden.
- After do surgery,please make a evaluate by your doctor if you can use the machine or not (We suggest use the machine after 3-6 months).Can use the machine after 3 months if eutocia ,after 6 months if cesarean.
- The machine can use for any health body part expect the Eyes ,Throat,Ears ,glottis and genital.
- Unless directed by a doctor, Please do not use the machine on ears ,Inside the mouth , neither to mucous membrane,nipples ,nevus.
- The epileptic,Psychopath are forbidden .

Remark:Please wear the massage cloth while using the magic curve

### 2. While using the Anti-cellulite, if there are the following cases , please do not use the machine

- If have cardiac pacemaker,Internal defibrillator, Forbidden
- If the operation body part have metal implant
- People who have skin cancer , or before cancer lesions
- People who have serious blood pressure
- People who during pregnancy or sucking period
- The operation part have bacteria or virus infection or opened wound
- The operation part have the permanent filler . or had Fat filled ,Plastic surgery within 3 months

### 3. Notes for use the machine

- Use the machine regularly, will make your body more health , Please do not stop use for a long time .If use the machine not following the right rules and precautions , The effecton will may not good as you want .If there is no effect after use the machine for several times ,or have any questions while using the machine , Please consult your doctor .There are some differenteffects

because of the age ,living habit (sport style , late sleeping ,diet ,take antibiotics ), ect;  
- People still can use the machine if cure from the plastic surgery . But please consult the doctor .  
The machine only operated by professional

## 9. Operation cautions

---

- (1)The machine only for beauty salon , not for home use .The operator should be professional,The Non-profess operate will give rise to unnecessary hurt (The children andthe mentally disabled people are forbidden to operate )
- (2)Please do not disassemble the machine without our technician's guide or without the user manual giude
- (3)The machine should keep away from the water.when install the machine , please keep your hands dry
- (4)Please contact with the technician if the following occurs:
  - The liquid splashed on the instrument or get wet in the rai
  - The instrument have peculiar smell, Smoking or unnormal voice
  - The power wire broken
  - The instrument fall down or casing damage
- (5)Please do not put anything on the power wire , please keep away from the place which power wire will by stepping on
- (6)For safety , please turn off the instrument after use
- (7)Please do not insert anything others in the port , otherwise it may make fire or electric shock.If there are something fall inside of the instrument and you can't take them out, Please turn off the instrument , and contact with your local distributor or contact with us . (8)Please don't put the machine on the unstable trolley,Floor,shelf or car ,f fall down,the machine will be in serious broken.
- (9)To extent the instrument life ,please Turn on/off intercal time more than 1 minute (10)The voltage should in step with the regulation,otherwise the machine will may not work even broken the machine .
- (11)If the power wire broken , please contact with the seller
- (12)Our company reserves the right to modify the information in this introduction before bnnotify! And keep the explanations right for the above.

## Notes

---

## Contact

---

**VacuActiv**

### Contact

**contact@vacuactiv.com**

**+48 660 216 370**

#### **Available at**

Central European Time 9am - 5pm

Monday - Friday

### Technical support

[www.vacuactiv.com/technical-support/](http://www.vacuactiv.com/technical-support/)

[support@vacuactiv.com](mailto:support@vacuactiv.com)

**+48 501 186 938**

Modrzewiowa 1 Street, 7

6-251 Losino, POLAND

Phone +48 59 843 43 43

***VacuActiv*** | WORLD LEADING  
MANUFACTURER

**[vacuactiv.com](https://vacuactiv.com)**